

Twin-O-Vac Filters


If a Twin-O-Vac filter becomes wet, it is a safety design of the unit that the filters occlude and consequently the suction unit ceases to function. To rectify this problem, the filter must be replaced.

A filter consists of one *fine* hydrophobic filter (the thin one) and one *coarse* solid filter (the thicker one).

Both of these filters are **WHITE**. The blue pieces of paper you may receive with your filters are to protect the fine filter during storage or transport and they should be discarded and not fitted into the Twin-O-Vac.

When fitting the filters, the *fine* filter should be placed in first (against the wire gauze screen) and the *coarse* filter then placed on top of the *fine* one.

These filters are made to occlude when they are wet. The Twin-O-Vac should not therefore be stored or left with any liquids in the jar. As mentioned above, after use the jars should be removed, cleaned (sterilised if necessary) and *completely dried* before being refitted to the Twin-O-Vac.

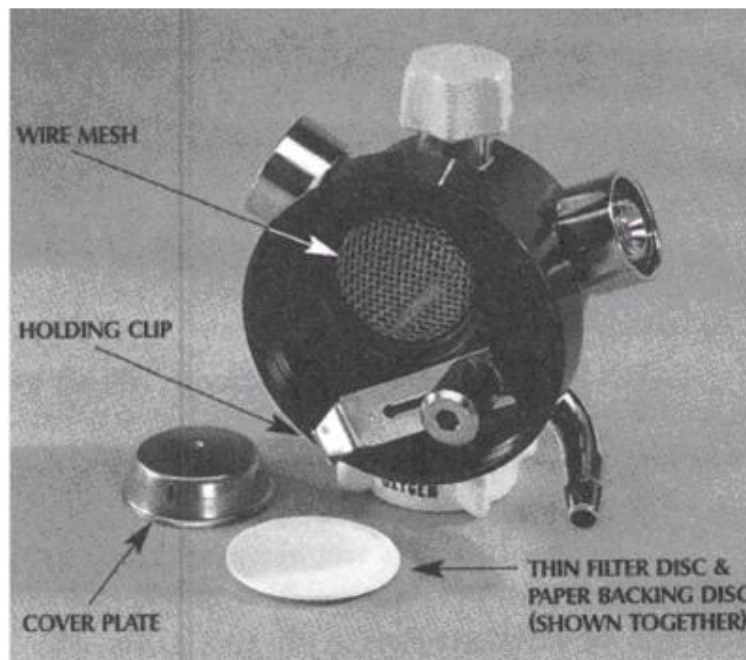

Please contact our office at the above numbers if you have any further queries.